

**Tourni
STRETCH**


**Tourni
STRETCH SP**

Podveze za jednokratnu
upotrebu za odrasle i djecu

Upute za korištenje



1

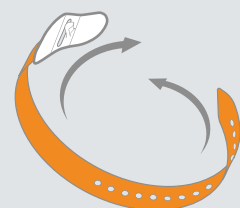

Provucite TourniSTRETCH oko
udova s klinom prema gore

2




Provucite rep kroz kopču i
zategnite ga u skladu s tim

3




Povucite rep unatrag preko
klina da biste ga učvrstili

4

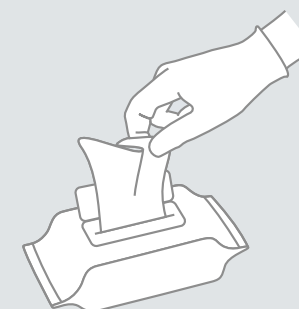


Izvadite jednu maramicu iz
pakiranja i potpuno je otvorite

5

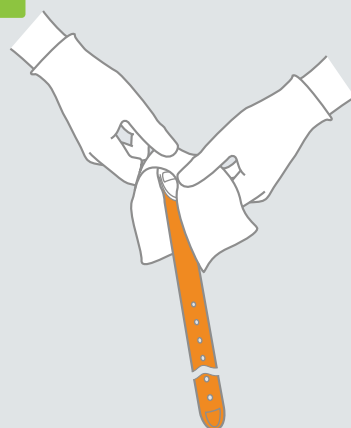

Očistite kopču maramicom

6

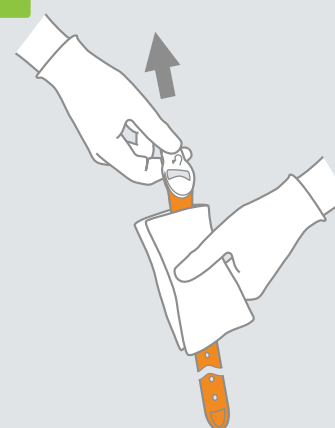


Držite kopču i povucite
TourniSTRETCH kroz krpu od
čistog do prljavog

7



Ostavite TourniSTRETCH da se
osuši na zraku*

8

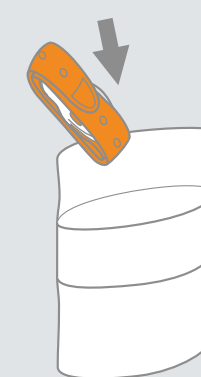

Stavite TourniSTRETCH u
vrećicu za pohranu za jednog
pacijenta za ponovnu upotrebu



HR



asephhealthcare.com



*Clinell univerzalne maramice učinkovite su protiv većine organizama
unutar 10-30 sekundi

Za preporučenu upotrebu pogledajte smjernice Clinell Universal Wipe

**ASEP
HEALTHCARE**

Rev 1 HR