


**Tourni  
STRETCH**

**Tourni  
STRETCH SP**

Tourniquets Úsáide Othair Aonair  
d'Úsáid Aosach & Péidiatraiceach  
Treoracha le haghaidh úsáide



**1**

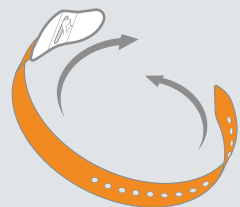

Pas TourniSTRETCH thart ar  
thaobh an phionna géag suas

**2**




Feed eireaball trí búcla agus  
teannas dá réir sin

**3**




Tarraing eireaball ar ais thar an  
bpionóg chun an áit a dhaingniú

**4**

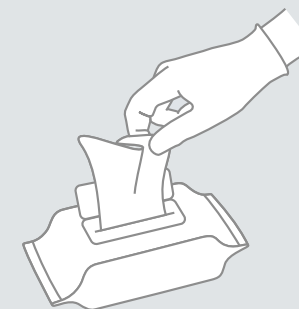


Bain wipe amháin as an phacáiste  
agus unfold go hiomlán

**5**

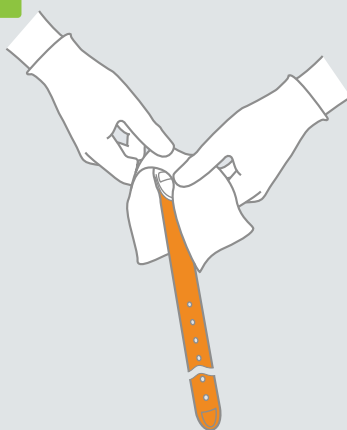

Glan an búcla leis an wipe

**6**

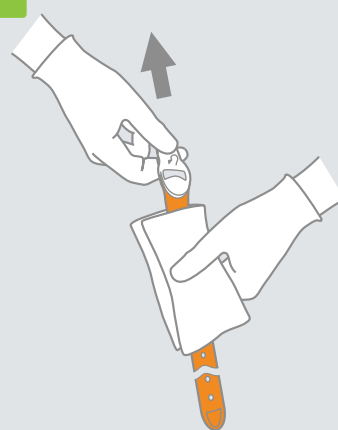


Coinnigh búcla agus tarraing an  
TourniSTRETCH tríd an wipe ó  
ghlan go salach

**7**



Fág an TourniSTRETCH chun  
aer tirim\*

**8**

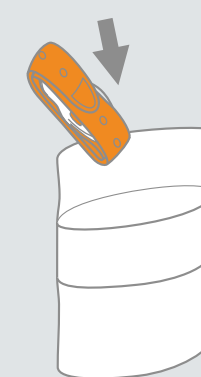

Cuir TourniSTRETCH i Mála Stórála  
Othar Singil lena athúsáid



IE



[asephealthcare.com](http://asephealthcare.com)



\*Tá Clinell Uilíoch Wipes éifeachtach i gcoinne fhormhór na  
n-orgánach laistigh de 10-30 soicind

Déan tagairt do Threoirlínte Clinell Uilíoch Wipe maidir le húsáid mholta

**ASEP  
HEALTHCARE**

Rev 1 IE